

THE JOURNAL OF PROFESSIONAL EXCELLENCE

# Dimensions<sup>®</sup>

OF DENTAL HYGIENE

## Transmitting TB

Separating the Facts from the Hype

Blood Sugar Testing In  
the Dental Operator

Practice In Comfort

Automatic Instrument  
Sharpening

Interdental Cleaning  
Devices



**Dimensions  
Goes Live!**  
Watch for  
a Partners in  
Practice Live  
CE Event in  
Your City...

September 2007

A Peer Reviewed Journal

WWW.DIMENSIONSOFDENTALHYGIENE.COM

VOLUME 5, NUMBER 9

# HurriSeal<sup>®</sup> Dentin Desensitizer

Made by Beutlich<sup>®</sup> LP Pharmaceuticals

## Specs:

HurriSeal is a HEMA-based dentin desensitizer that also contains sodium fluoride and benzalkonium chloride. The desensitizer is transparent, requires no mixing or light curing, and leaves no film thickness. HurriSeal is versatile as it can be used to treat existing cervical sensitivity or prevent sensitivity associated with restorative procedures.

HurriSeal treats existing cervical sensitivity with 6-9 months of effectiveness and is fast and simple to apply. Application time is approximately 20 seconds and results are immediate. Patients may be reimbursed for this procedure, as dentin desensitization is an American Dental Association-recognized procedure and may be billed under insurance.

HurriSeal prevents sensitivity when used under restorations and is ideal for class 5 sensitivity. To prevent sensitivity commonly associated with temporary restorations, HurriSeal can be applied to crown preps prior to provisionalization. When used as a wetting agent during restorative procedures, HurriSeal shows increased bond strength and lasts the lifetime of the restoration.

HurriSeal may be applied to treat cervical or root sensitivity in a simple chairside procedure or placed prior to restorations, helping prevent any post-operative sensitivity. HurriSeal does not contain glutaraldehyde, a common ingredient in desensitizing products, that can be irritating to soft tissue. Recommended for sensitivity associated with tooth erosion, gingival recession, tooth whitening, exposed root surfaces following periodontal treatment, and temporary and permanent restorations.

## Unique Features:

HurriSeal is simple and easy to use; no mixing or light curing is needed. A chairside application provides immediate results that last from 6 to 9 months for cervical treatment.

When applied during restorative procedures, HurriSeal will prevent sensitivity for the lifetime of the restoration. A quick 20 second application of HurriSeal after etching and before applying a bonding agent is all that is needed.

- HurriSeal is packaged in a 12 ml bottle
- Soft tissue friendly
- Patented formula
- Can be used alone or in conjunction with restorative materials
- Price—HurriSeal is very economical
- Simple to use
- Unconditional guarantee



## How to order:

HurriSeal is available through dental dealers or through Beutlich directly.  
 Beutlich LP Pharmaceuticals  
 1541 Shields Dr  
 Waukegan, IL 60085  
 (800) 238-8542  
[www.beutlich.com](http://www.beutlich.com)  
[beutlich@beutlich.com](mailto:beutlich@beutlich.com)

HurriSeal is a registered trademark of Beutlich LP Pharmaceuticals.

